# VERMEIREN

## 201 Delta

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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## Content

<b>Content</b> .....	<b>1</b>
<b>1 Preface</b> .....	<b>1</b>
<b>2 Your product</b> .....	<b>2</b>
<b>3 Before use</b> .....	<b>2</b>
<b>4 Assembly/disassembly and adjustments</b> .....	<b>3</b>
<b>5 Using your rollator</b> .....	<b>5</b>
<b>6 Maintenance</b> .....	<b>6</b>
<b>7 Technical specifications</b> .....	<b>7</b>

the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 1 Preface

Congratulations! You are now owner of a Vermeiren rollator!

Thank you for your trust in the products of Vermeiren. To support you on the use of this rollator and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your rollator.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### **Important note**

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### **Information available**

On our website, <http://www.vermeiren.com/> , you will always find the most recent version of

## 2 Your product

Figure 1 Rollator components

1. Handgrip
2. Brake lever
3. Tray
4. Basket
5. Storage bag
6. Front wheel
7. Rear wheel

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- rollatorframe with 3 wheels;
- height adjustable handgrips with brakes;
- shopping basket;
- plastic tray;
- bag;
- manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 3.2 Intended use

CAUTION

**Risk of falling**

**The rollator is meant as support of the leg function during walking. Do not use the rollator if you have other physical limitations that may cause unsafe use of your rollator, like a weak arm, balance disorders, ...**

The rollator is designed and produced solely to be used as walking aid for one person with a maximum weight of 125 kg.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide better stability, support and safety during walking.

The rollator can be used indoors and outdoors.

It is not intended for transport of goods and/or more people than the user.

It should only be used on surfaces where all wheels are touching the ground. Do not use the rollator on rocky, bumpy or sandy terrain, or places with a steep inclination. The rollator may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your rollator.

Please check all technical details and limits of your rollator in §7.

### 3.3 Safety instructions

 CAUTION Risk of injuries

Please read and follow the instructions in this manual. Otherwise you may get injured, or your rollator may get damaged.

- Check that no stones or other objects block the wheels of the rollator.
- Make sure that both brake levers are in parking mode before you use the rollator as aid to stand up. When you are standing comfortable and stable, you can unlock the parking brakes.
- While walking, keep both hands on the handgrips.
- Be aware that some parts of your rollator may get very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Before every use, check that the height adjustments are firmly secured, see §4.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

### 3.4 Symbols on the rollator

 Identification plate is located on the lower right, at the inner side of the base frame

 Maximum weight of user in kg

 Type designation

### 3.5 Shipping and storage

The shipping and storage of the rollator should happen according to the technical specifications in §7. Make sure that the rollator is stored dry.

Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the rollator to prevent damage.

## 4 Assembly/disassembly and adjustments

### 4.1 Assembly

#### 4.1.1 Mounting and adjusting the handgrips

1. Place the rollator on all wheels. Make sure that it is standing stable.
2. Place the handgrip (1) into the frame (2), and slide up/down until the desired height is reached (range of 100 mm, 5 positions).
3. Place the bolt through the metal bracket with hook, the frame tube and the handle tube on the inside of the frame.
4. Place the spacer and star knob (3), and tighten firmly.
5. Repeat for the second handgrip. Make sure that both handgrips are well secured, and that they are set to the same height.

To adjust the handgrips, remove the star knob and bolt, and repeat from step 2.

#### 4.1.2 Mounting the storage bag, basket and tray

 CAUTION Risk of falling or tipping over

**Do not overload the storage bag, basket or tray.**

The maximum carrying weight of the storage bag, basket and tray can be found in §7.

#### Storage bag

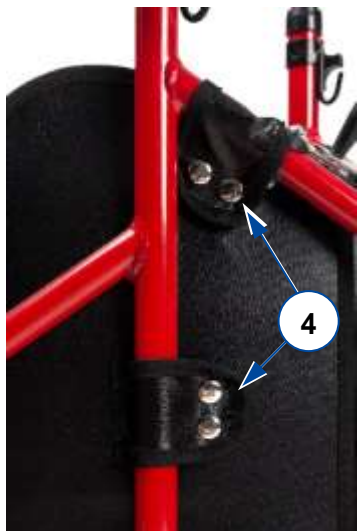
The storage bag is meant to be mounted on the inside of the lower frame.

1. Fold the rollator open.

2. Mount the storage bag on the frame tubes with the eight snap fasteners (4).

#### Basket

Mount the basket on the two hooks (5) below the handgrips. Make sure that the basket has been mounted centrally to avoid tipping over!

#### Tray

Place the tray on the top of the metal basket. Make sure that it fits well over the basket rims to avoid falling or tipping over.

## 4.2 Disassembly

CAUTION

**Risk of damage**

**Pay attention to the position of the brake cables. Don't pull the handgrips too high to prevent damage to the brake cable.**

The disassembly of the rollator is conducted in reverse order to the assembly:

1. Remove the tray and basket.
2. Storage bag: Open snap fasteners (4) and remove the bag.

3. Handgrip with brake lever (2x): Remove the star knob (3) and bolt, and pull the handgrip out of the frame.

## 4.3 Adjusting the brakes

The force of the brakes can be adjusted by loosening/tightening the screw (6) on the back of the brake lever.

## 5 Using your rollator

### 5.1 Brake system

 CAUTION Risk of falling

**Check the working of the brakes to ensure your safety. The condition of the brakes is influenced by wear, oil and other contamination of the rear wheels and braking mechanism.**

The rollator is equipped with brakes, operated by two brake levers (7). The brake levers operate in two modes: running brakes and parking brakes.

 CAUTION Risk of damage

**Do not push the rollator too hard while using the running brakes. This may cause damage to the tyres or brakes.**

- To use the running brakes, pull both brake levers (7) simultaneously upwards [A] in a squeezing motion. To stop braking, release the brake levers.
- To use the parking brakes, push both brake levers (7) downwards in locked position [B]. Always use both parking brakes simultaneously. To end the parking mode, pull both brake levers simultaneously upwards.

### 5.2 Unfolding / folding

 CAUTION Risk of entrapment

**Keep fingers, buckles, jewellery and clothes away from the folding mechanism.**

 CAUTION Risk of falling

**Check that the rollator is completely open and that the frame is locked.**

#### 5.2.1 Unfolding rollator

1. Place the rollator with all wheels on the ground while holding the frame tubes.
2. Pull the handgrips apart, and push the folding mechanism (8) as far forward as possible using the knob (9). Prevent your fingers from being trapped in the mechanism!
3. Mount the storage bag, basket and tray.

#### 5.2.2 Folding rollator

1. Remove the tray and basket.
2. Stand behind the rollator.
3. Empty the storage bag.
4. Pull the knob (9) of the folding mechanism (8) backward to close the frame a bit.
5. Close the frame completely by pushing the handgrips together.

## 6 Maintenance

**i** Regular care ensures that your rollator is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com)

### 6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the rollator. Avoid soaking.

Do not use abrasive cleaning agents to clean.

### 6.2 Disinfection

CAUTION

**Risk of damage**

**Disinfections may only be undertaken by trained persons. Consult your specialist dealer.**

### 6.3 Inspection / Service / Repairs

The lifespan of the rollator is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the rollator, it should be inspected on the following points:

- Completeness.
- Folding mechanism: functioning well.
- Bolts and nuts: tightly fastened.
- Brakes: functioning well.
- Wheels (stability, tracking, fixing screws).  
In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).
- Handgrips (safe locking of height adjustment system, good support of handgrips).
- Storage bag (wear or damage).

Your rollator is constructed of aluminium and plastic components. The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only

genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

### 6.4 Disposal at end of life

At end of life, you need to dispose of your rollator according to the local environmental legislation. The best way to do so, is to disassemble the rollator to facilitate the transport of recyclable parts.

## 7 Technical specifications

The technical details below are only valid for this rollator, at standard settings and optimal ambient conditions. Take these details into account during use.

<b>Brand</b>	<b>Vermeiren</b>
<b>Type</b>	<b>Rollator</b>
<b>Model</b>	<b>201 Delta</b>

<i>Description</i>	<i>Dimensions</i>
Maximum user weight	125 kg
Maximum weight of storage bag	5 kg
Overall length	575 mm
Folded length	610 mm
Overall width	680 mm
Folded width	270 mm
Overall height	820-920 mm
Total mass - without bag, basket and tray	5,8 kg 4,55 kg
Wheel diameter	190 mm
Brakes	Running & parking brakes
Max. inclination forwards	15°
Max. inclination backwards	7°
Max. inclination sideways	3,5°
Storage temperature	+5°C - +41°C
Storage humidity	30% - 70%

We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP NV  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### All rights reserved, including translation.