


Battery pack complete


1	BATTERY CEL LITHIUM	1914105
2	MIDDLE PART HANDLE	1914107
3	FRONT PART HANDLE	1914109
4	SCREW M5	1914172
5	SCREW M3x10	1914173
6	NUT M3	1914174
7	CHARGING SOCKET	1913963
8	BATTERY BOX COVER	1914110
9	BATTERY BOX LOWER PART	1914111
10	SCREW M4x10	1914175
11	SCREW M3x13	1914176
12	BATTERY BOX LOCK	1914121
12	KEY FOR BATTERY BOX LOCK	1914825
15	LITHIUM CHARGER 2 AMP	1914124
16	CHARGER POWER CORD	1914125

ARMREST SET

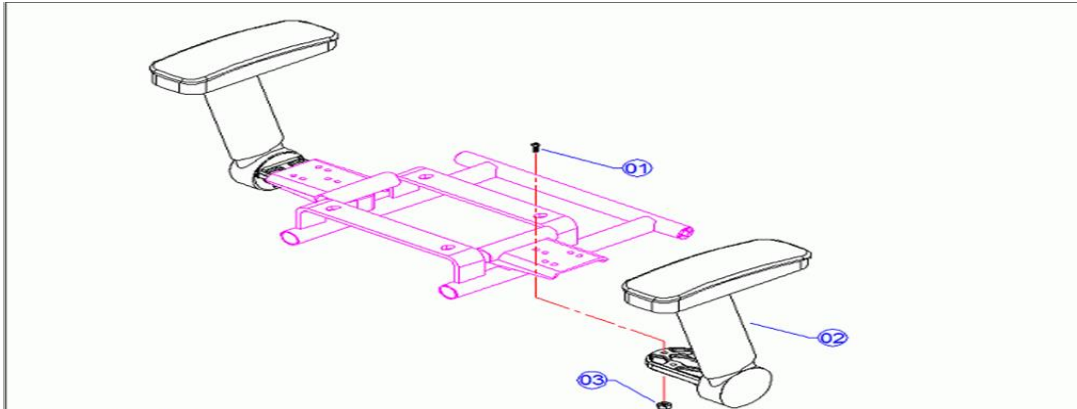

1	BOLT HEXAGONAL M6x20	1995308
2	ARMREST LEFT	1913937
	ARMREST RIGHT	1913938
3	NUT M6	2990533


REAR WHEEL COMPLETE

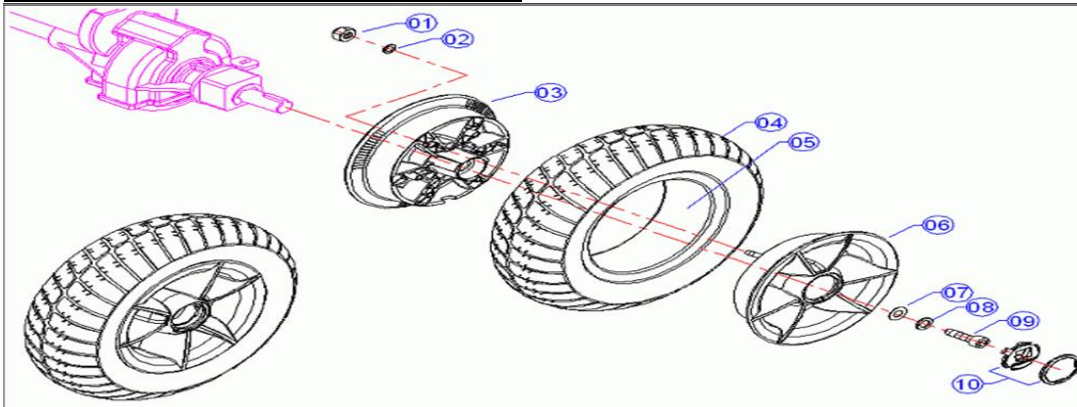


1	NUT M6 INOX	2990533
2	SPRING WASHER M6	1004414
3	INNER RIM	1913939
4	TIRE BLACK	1911594
5	INNER TIRE	1995114
6	OUTER RIM	1913940
7	WASHER	1913941
8	SPRING WASHER M6	1913942

9	SCREW M8x20	1913959
10	WHEEL CAP	1913960
11	AXLE KEY	1918922

ANTI TIPPING WHEEL

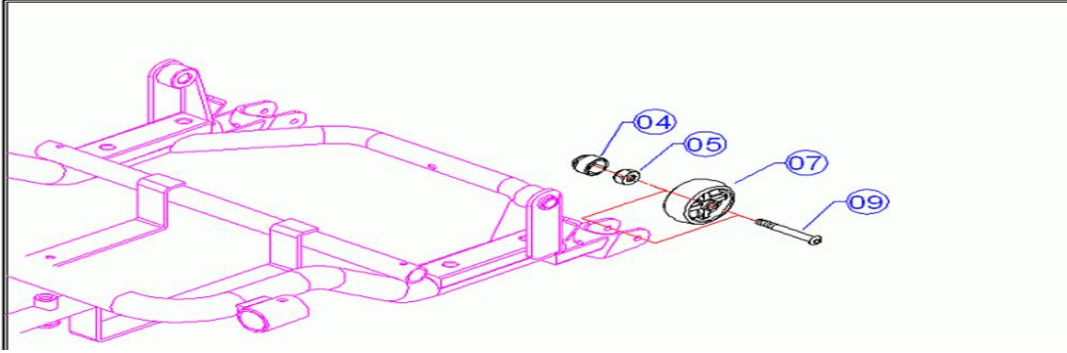


4	NYLON CAP_M6	
5	NUT M6 INOX	1995312
7	ANTI TIPPING WHEEL	1915522
9	BOLT M6 x40 INOX	1995305

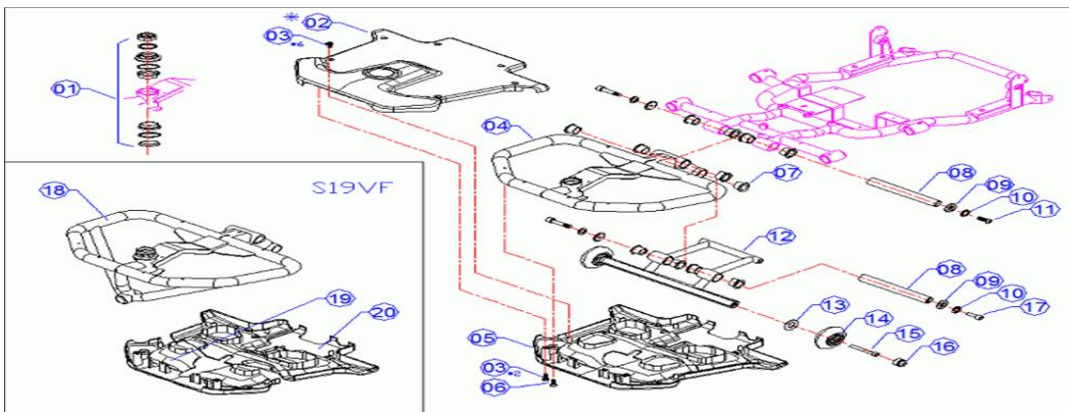

1	BEARING SET	1913961
2	FRONT FRAME TOP COVER BLUE	1913966
2	FRONT FRAME TOP COVER RED	1913967
5	FRONT FRAME LOWER COVER	1914122


12	Set transport wheels +support	1919572
13		
14		
15		
16		

REAR FRAME

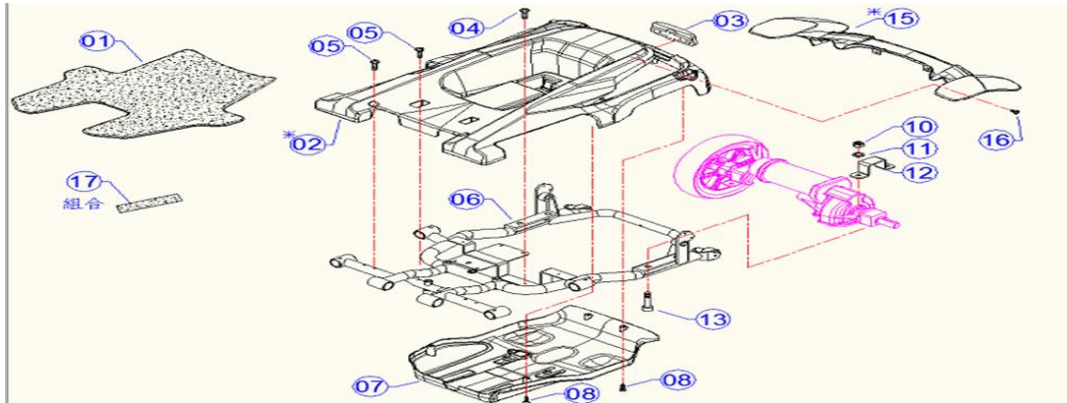


1	CARPET	1913962
2	REAR COVER BLUE	1914178
2	REAR COVER RED	1914179
7	UNDER BODY	1914148
15	REAR COVER + MUDGUARD	1914149

FOLDING SYSTEM

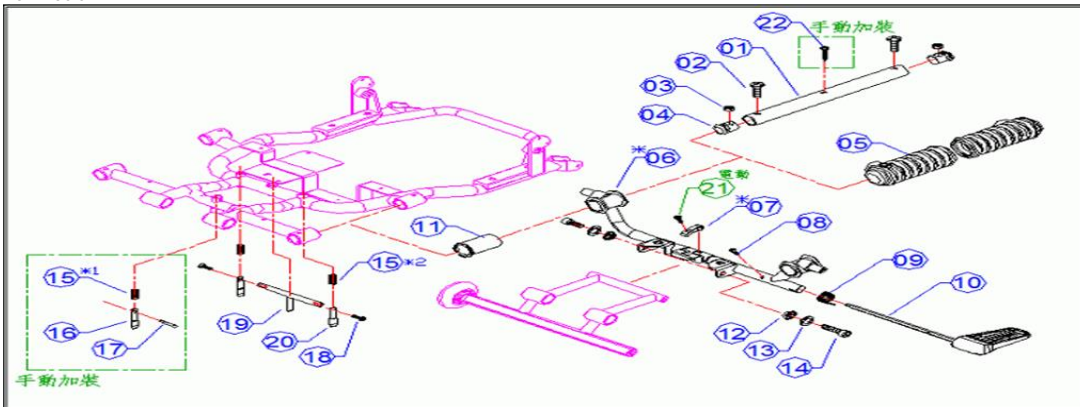


5	RUBBER DUST COVER	1914150
6	PEDAL AXEL	1914203
7	LOCKING PIN PEDAL	1914204
8	BOLT M4x10	1914205
9	SPRING PEDAL AXEL	1914206
10	PEDAL	1914127

HANDEL BARS


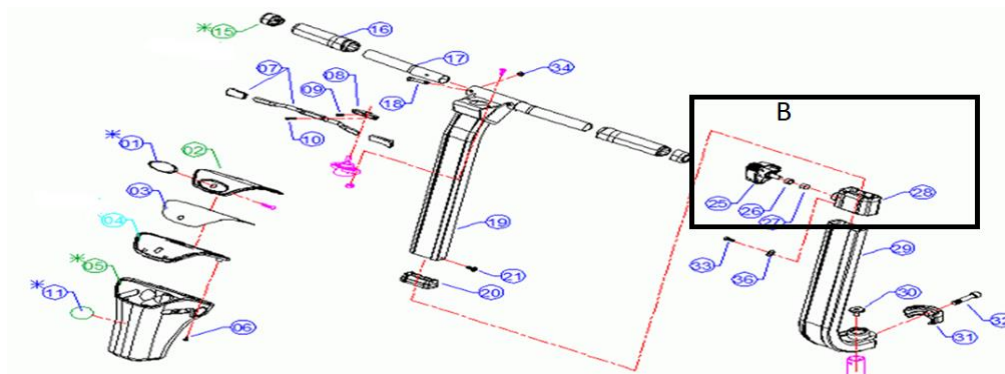


29	LOWER TUBE HANDLE BAR		1920041
B	SE*STUURVERST.KOMPLEET MET STERKNOP		1915968
5	HANDLE BAR COVER RED		1914858
5	HANDLE BAR COVER BLUE		1914857
7	SPEED LEVER		1914207
8	SPEED LEVER ATTACHEMENT		1914208
not depicted	COVVER KNOB		1915992
27	BLOCKING WASHER		1918062


SEAT ASSEMBLY

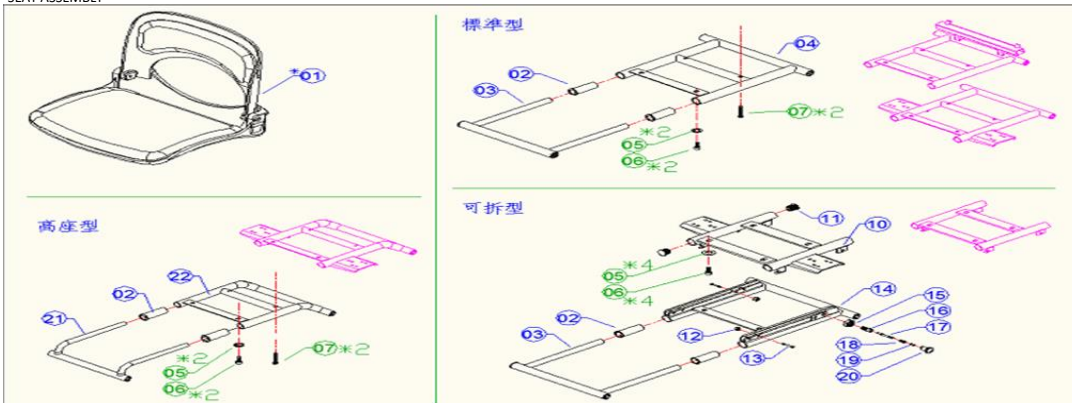


SEAT FOLDING SYSTEM

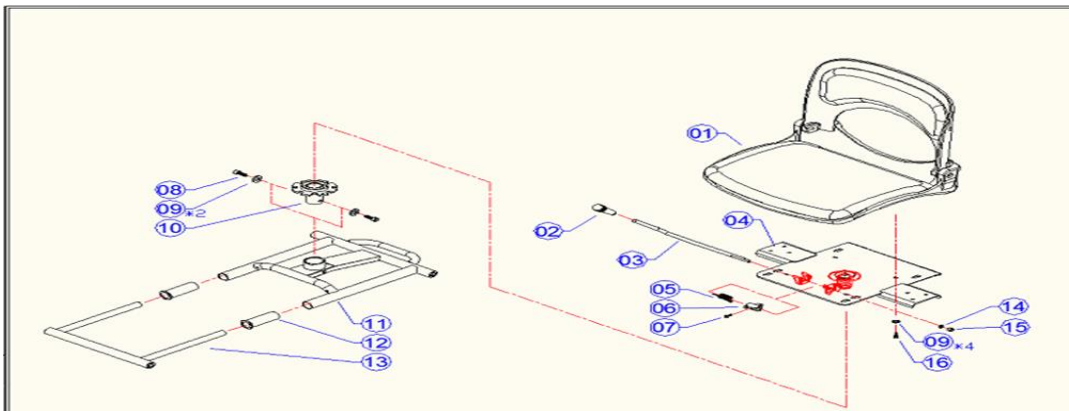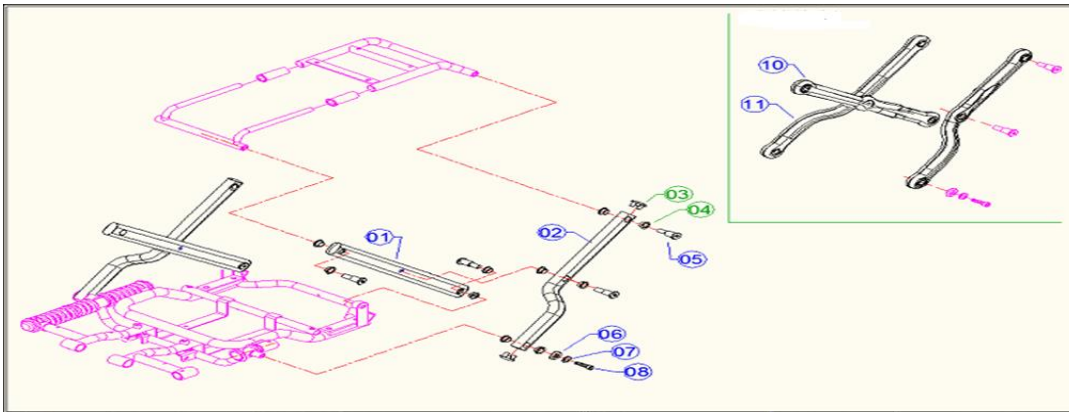


SWIVEL SEAT		
10	SEAT TUBE	1917958

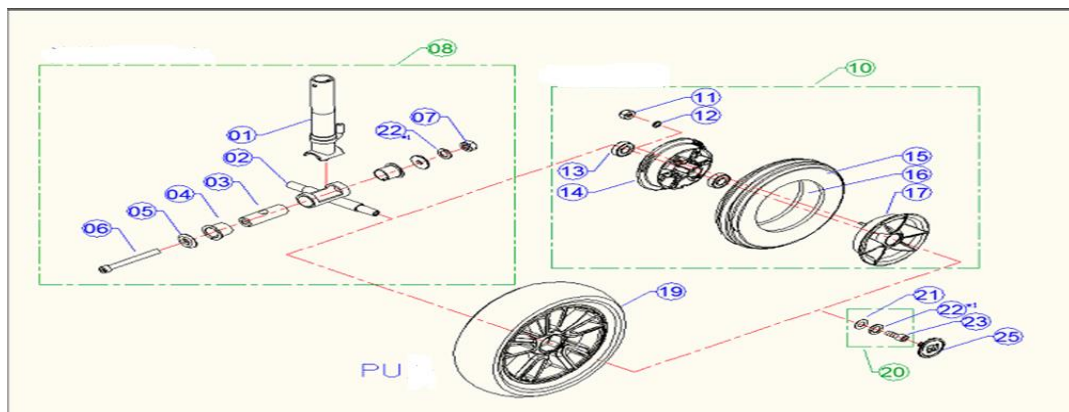

FRONT AXLE COMPLETE		
1	FRONT AXLE TUBE	1920033
2	FRONT TURNING AXLE	1914154
3	CENTER TUB	1914209
4	BEARING SHAFT	1914210
10	COMPLETE WHEEL	1915126
14	INNER RIM	1914155
16	INNER TUBE	1001263
17	OUTER RIM	1914156

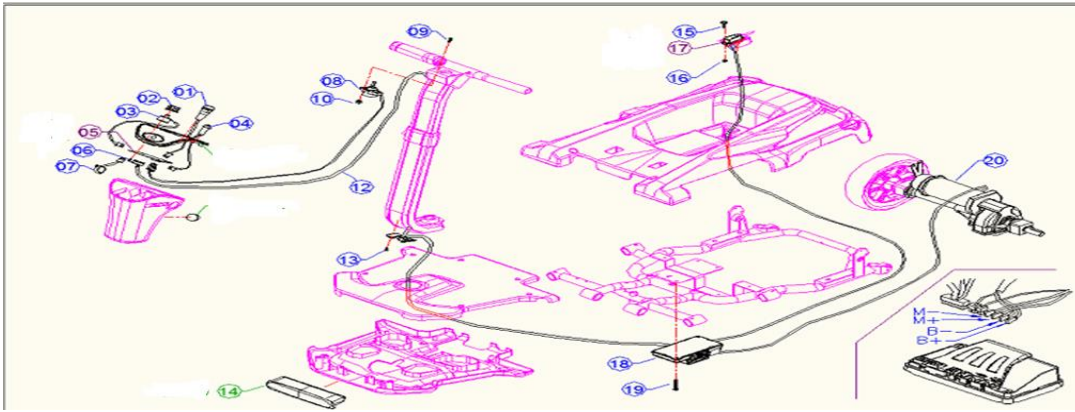

1	LOCK	1914160
1	KEY	1914825
2	SPEED POT KNOB	1914213
3	SPEED POT	1914161
4	HORN	1914166
5	PCB TILLER	1914167
7	DRIVE POT	1914168
9	SCREW M4x15	1914211
10	NUT M4	1914212
18	CONTROLLER DYNAMIC	
20	MOTOR +BRAKE+ DIFF	1914255

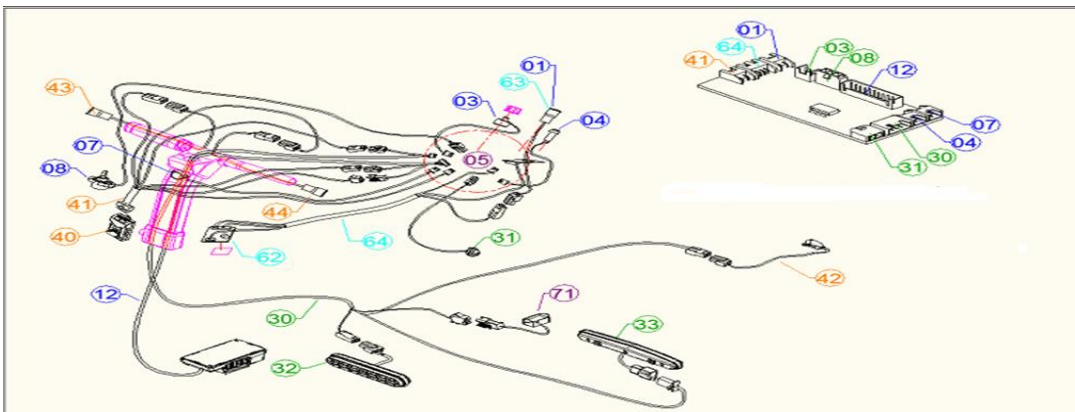

12	MAIN CABLE			1914169
31	CABLE +LED			1914171